Flamigel® Repair + Protect Hand Gel

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Flamigel® Repair + Protect Hand Gel

What is Flamigel® Repair + Protect Hand Gel?

Flamigel® Repair + Protect Hand Gel is a smooth, fragrance-free and quick absorbing barrier gel for application on irritated, damaged, cracked or chapped hands.

What is Flamigel® Repair + Protect Hand Gel used for?

Flamigel® Repair + Protect Hand Gel provides a moist mechanical barrier for irritated, damaged, cracked or chapped hands. This means that Flamigel® Repair + Protect Hand Gel helps to nourish and repair damaged hands.

How to use Flamigel® Repair + Protect Hand Gel?

Unscrew the tube cap, remove the tamper seal and close the tube after use. Apply Flamigel® Repair + Protect Hand Gel gently to your hands as often as necessary.

How does Flamigel® Repair + Protect Hand Gel work?

Flamigel® Repair + Protect Hand Gel repairs, protects and nourishes the skin barrier of damaged and irritated hands. The cooling gel soothes cracks or chaps on the hands. The gel leaves an invisible protective barrier on the skin. Flamigel® Repair + Protect Hand Gel is non-greasy and non-sticky. Flamigel® Repair + Protect Hand Gel contains arginine, a building block of the skin.

Special precautions when using Flamigel® Repair + Protect Hand Gel

- Flamigel® Repair + Protect Hand Gel is suitable for external use only, do not ingest.
- Do not use Flamigel® Repair + Protect Hand Gel if there is a known allergic reaction to any of the ingredients. Do not use Flamigel® Repair + Protect Hand Gel on the eyelids or in the eyes. If Flamigel® Repair + Protect Hand Gel comes into contact with the eyes, rinse the eyes with running water and consult a doctor or nurse.

Reporting of side effects

Flamigel® Repair + Protect Hand Gel is a medical device: if any serious incident occurs in relation to the device, or your condition worsens, please contact your doctor or pharmacist. This incident should be reported to Flen Health and the competent authorities of your country (the distributor, see below, can give you details of who this is). By reporting an incident, you can help provide more information on the safety and the efficacy of this medical device.

Storage conditions

- Store at room temperature (< 25 °C), in a dry place and in the original packaging.
- When properly stored and closed after each usage, Flamigel® Repair + Protect Hand Gel can be used until the expiry date (USE BY year/ month), which is the last day of the indicated month and year. Do not use Flamigel® Repair + Protect Hand Gel after the expiry date.

Ingredients

Hydrocolloid, Arginine, Purified water, Macrogol, Branched Chain Fatty Acid (BCFA), Methyl-p-hydroxybenzoate (E218), Propyl-p-hydroxybenzoate (E216), Disodium EDTA.

How supplied

Tubes of 50g

Manufacturer

Flen Health NV/SA, Blauwesteenstraat 87, B-2550 Kontich, Belgium
www.flenhealth.com – info@flenhealth.com
MADE IN BELGIUM

Distributors

BE: Flen Health NV, Blauwesteenstraat 87,
2550 Kontich, Belgium

LU: Flen Health SA, House of BioHealth, 29, Rue Henri Koch,
4354 Esch-sur-Alzette, GD Luxembourg

Dispositif médical
Medisch hulpmiddel
Medizinprodukt
Medical device

Date of issue: 2021/02

0905-959-A

I182v2